F

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E

PACKAGE INFORMATION LEAFLET

Aktimed GRID TAPE PRO is a grid-shaped tape based on polyurethane that has a 10% synthetic CBD-solution integrated into the acrylic adhesive. Contains no THC and therefore does not have an intoxicating effect. The focus is on the physical and mechanical properties of the adhesive tape.*

APPLICATIONS

- Trigger Point and Acupuncture Point Taping

INDICATIONS FOR USE

- The product is intended for single use.
- Application is recommended by a trained person such as doctor, naturopath or physiotherapist.
- Before applying the grid tape degrease the skin.
- The tape should be applied 20-30 minutes before physical activity.
- It may occasionally cause skin irritation. In this case, immediately remove the tape.

SAFETY INFORMATION

- In case of allergy to any of the ingredients do not use the tape or first test on a small area of skin!
- Do not adhere to open wounds or skin injuries!
- In the case of skin reactions such as itching or redness remove the tape immediately!
- Pregnant women may only apply the tape after consultation with the attending physician!
- Keep out of reach of children!

E

APPLICATION TIME

The tape lasts up to 6 days on the skin.

INGREDIENTS

80% polyester, 15% polyurethane, 5% acrylate. Adhesive mass contains a 10% synthetic CBD-solution.

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Store in a dry and cool place away from sunlight. Store between 5° C and 30° C.

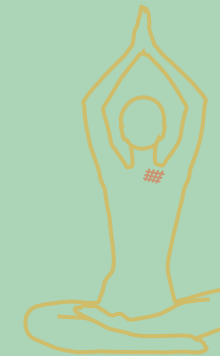
* Effects have not been scientifically proven

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Aktimed GmbH, Theaterstr. 18, 69117 Heidelberg, Alemania